

# MINI-BAG PLUS Container System

## MINI-BAG PLUS system directions:

Only for Single Dose Powdered Drug Vials with 20 mm Closures

Use Aseptic Technique

<p><b>1</b></p>	<p><b>2</b></p>	<p><b>Assembly</b></p> <ol style="list-style-type: none"> <li> <ul style="list-style-type: none"> <li>Remove vial cover</li> <li>Disinfect stopper</li> </ul> </li> <li> <ul style="list-style-type: none"> <li>Peel off foil cover</li> <li>Inspect adaptor for moisture Discard if found</li> </ul> </li> <li> <ul style="list-style-type: none"> <li>Place vial upright</li> <li>Hold firmly</li> <li>Push adaptor down until vial snaps in place</li> <li><b>DO NOT TWIST</b></li> <li>Pull vial to ensure fully seated</li> </ul> </li> </ol>
<p><b>3</b></p>	<p><b>4</b></p>	<p><b>Reconstitution</b></p> <ol style="list-style-type: none"> <li> <ul style="list-style-type: none"> <li>Squeeze bag and check vial</li> <li>Use only if vial fully seated and dry</li> <li>Bend up then down to break seal</li> </ul> </li> <li> <ul style="list-style-type: none"> <li>Hold bag with vial down</li> <li>Squeeze solution into vial until <b>half full</b></li> <li>Shake to suspend drug in solution</li> </ul> </li> <li> <ul style="list-style-type: none"> <li>Hold bag with vial upside down</li> <li>Squeeze bag to force air into vial</li> <li>Release to drain suspended drug from vial</li> <li>Repeat steps 5 and 6 until vial is <b>empty</b> of drug and solution is thoroughly mixed. <b>Ensure drug is completely dissolved. Do Not Remove Drug Vial.</b></li> </ul> </li> </ol>
<p><b>5</b></p>	<p><b>6</b></p>	<ol style="list-style-type: none"> <li> <ul style="list-style-type: none"> <li>Remove port protector.</li> <li>Attach administration set per its directions.</li> </ul> </li> <li> <ul style="list-style-type: none"> <li>Hang container on I.V. pole and prime set per directions. Ensure that vial is empty of drug and solution. Repeat step 6 if drug and solution remain in vial.</li> <li><b>Warning:</b> Do not use in series connections</li> </ul> </li> <li> <ul style="list-style-type: none"> <li>Administer medication per directions. Use within specified time for drug stability. Refer to drug package insert.</li> </ul> </li> </ol>

Rx only.

To ensure safe and proper use of this device, refer to the complete instructions for use.

To Order Baxter Products, call 1-888-229-0001



Options at work.

MEDICATION DELIVERY