



For use on a **swab-able luer access valve** as a disinfecting cleaner prior to line access and to act as a physical barrier to contamination between line accesses.

# SwabCap™

Use aseptically.

Follow Excelsior Medical's instructions for use.


**FOR SINGLE USE ONLY**



LUER ACCESS VALVE  
DISINFECTION CAP  
Contains 70% IPA  
**SINGLE USE ONLY**

Latex Free  
DEHP Free  
Preservative Free

## Peel.

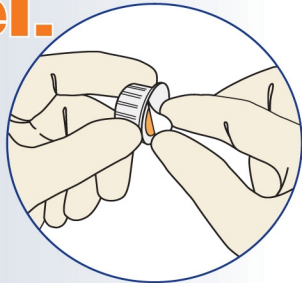

Remove protective cover by pulling tab. **Do not remove SwabCap™ from white plastic holder.**

## Twist.




Hold luer access valve in one hand while pushing and twisting the **SwabCap™** in a clockwise direction onto the luer access valve.

## Protect.



After **SwabCap™** has been twisted into place, gently pull white plastic holder off luer access valve.

## To Remove for access to valve

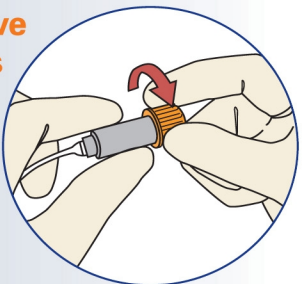

**Two-handed removal must be used to make sure luer access valve is not removed unintentionally.**

Grasp luer access valve and twist **SwabCap™** counter clockwise away from valve. Remove and discard. **Inspect luer access valve to verify that it's connected securely to the catheter.**

Valve is ready for access.

**No further swabbing is necessary after removal.**

Distributed by:



**WOLF MEDICAL SUPPLY, INC.**

For more information:  
800.335.9653